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Version 13 - Approved changes

Addition is highlighted and the removal is stricken through

**Respirator Fit Testing**

**NOTES:**

- CSC has confirmed with the fit testing machine (Portacount) manufacturer (TSI Inc.) that in the context of the COVID-19 pandemic, it is safe to undertake fit testing if appropriate hygiene measures are followed. The tubes do not need to be changed or cleaned after each use. The Portacount uses two tubes – one that takes an air sample from the ambient air, and one that aspires from inside the mask during the fit test. The air flow is under constant negative pressure (air towards the machine and not the user) and no air is returned inside the mask. Air moving towards the Portacount must travel through a condenser which is composed of a wick soaked in 99% pure alcohol as well as two HEPA filters. TSI has developed [FAQ Portacount Use during Pandemic](#).
- ~~For the duration of the COVID-19 pandemic, there will be exceptions for N-95 fit testing due to the fact that the fit test procedure itself consumes a respirator. The N-95 respirators are in short supply and must be preserved and conserved for situations requiring their use (e.g. aerosol-generating medical procedures). Periodic fit testing for N-95 respirators may be reinstated, if an adequate supply of N-95 respirators is identified and supplied to CSC. CSC health services is maintaining a PPE inventory, including respirators and will report if / when CSC obtains such a supply.~~

	Low Risk – Green	Low-Moderate Risk Mitigation Strategies - Grey	Moderate Risk Mitigation Strategies - Yellow
Fit Testing Planning	<ul style="list-style-type: none"> <li><input type="checkbox"/> Each respirator user at CSC must be fit tested prior to using their respirator and a minimum of every 2 years thereafter.</li> <li><input type="checkbox"/> The fit test applies to the individual and their specific make/model/size of respirator. The user must ensure that they follow the donning instructions of the specific respirator to maintain a tight seal. The individual must keep track of the respirator(s) for which they have successfully been fit tested for and use this particular respirator while being clean shaven, where applicable.</li> <li><input type="checkbox"/> Fit testing is typically performed locally by a person who has been trained or instructed on how to perform a quantitative fit test.</li> </ul>	<ul style="list-style-type: none"> <li><input type="checkbox"/> Fit testing frequency for SCBA / gas masks will be modified to biennial frequency (every two years). This aligns with the requirements of <a href="#">CSA Standard Z94.4-18</a> and the <a href="#">Canada Occupational Health &amp; Safety Regulations</a>. The reduced frequency reduces the amount of contact between fit tester and employees requiring fit testing.</li> <li><input type="checkbox"/> <del>For the duration of the COVID-19 pandemic, fit testing for N-95 respirators will be limited primarily to new employees and recruits (new respirator users).</del></li> <li><input type="checkbox"/> <del>Current</del> Employees should refer to their most recent fit test, as the employee’s fit test is only valid for the specific make, model and size of the respirator they were fit tested for. <b>If new N95s are received at the site, fit testing will be required for the new model.</b></li> </ul>	

	Low Risk – Green	Low-Moderate Risk Mitigation Strategies - Grey	Moderate Risk Mitigation Strategies - Yellow
		<ul style="list-style-type: none"> <li><input type="checkbox"/> If an employee has undergone recent significant changes that may affect their facial seal (e.g. weight changes, facial changes) they should report this to their supervisor and an ad hoc test may be arranged.</li> <li><input type="checkbox"/> The AWO will monitor compliance with the above standards, in consultation with the Workplace Health &amp; Safety Committee.</li> <li><input type="checkbox"/> Fit testers will need to be made aware of the infection prevention and control measures listed in this document prior to performing testing.</li> </ul>	
Fit testing procedure	<ul style="list-style-type: none"> <li><input type="checkbox"/> Regular fit testing procedures.</li> </ul>	<ul style="list-style-type: none"> <li><input type="checkbox"/> Fit testing procedure should be limited to one person at a time being in the room with the fit tester. If multiple Portacount machines are used simultaneously, they will be used in separate rooms.</li> <li><input type="checkbox"/> Physical distancing of 2 metres must be maintained at all times between the fit tester and the employee or recruit and the fit testing machine.</li> <li><input type="checkbox"/> Fit tester must wear a non-medical mask at all times while in the presence of an employee or recruit.</li> <li><input type="checkbox"/> The fit tester must explain the safety rules and requirements (e.g. physical distancing) to the employee or recruit undergoing the fit test.</li> <li><input type="checkbox"/> The fit tester must clean and disinfect the gas mask and SCBA after each fit test in accordance with the <a href="#">MSA Respirator Cleaning Guidance</a>.</li> <li><input type="checkbox"/> The fit tester must clean and disinfect the surfaces for which the employee or recruit undergoing the fit test has come in contact (e.g. desk, chair, exterior of mask, exterior of tube).</li> <li><input type="checkbox"/> The employee/recruit must also disinfect the gas mask and SCBA with a disinfectant wipe prior to donning the respirator.</li> <li><input type="checkbox"/> The fit tester and the employee or recruit must wash their hands before and after performing / undergoing the fit test procedure.</li> </ul>	

	Low Risk – Green	Low-Moderate Risk Mitigation Strategies - Grey	Moderate Risk Mitigation Strategies - Yellow
		<input type="checkbox"/> For those employees or recruits who N-95 masks are one time use only and are damaged by the probe during fit testing and must therefore be disposed of by the employee or recruit in a no touch receptacle after their fit test.	

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